

KN95 respirator - filtration efficiency 99%*

*According to the laboratory results at the Research Institute of Occupational Safety - ZL, Prague 15. 4. 2020

Respiratory resistance test:

low measured values indicate the possibility of easy breathing

>>> **High user comfort**

Determination of aerosol penetration:


average measured value 0,94%

>>> **Meets the requirements for FFP3 class respirators**



REF: 1320100792

Packing: 20 pcs/pkg

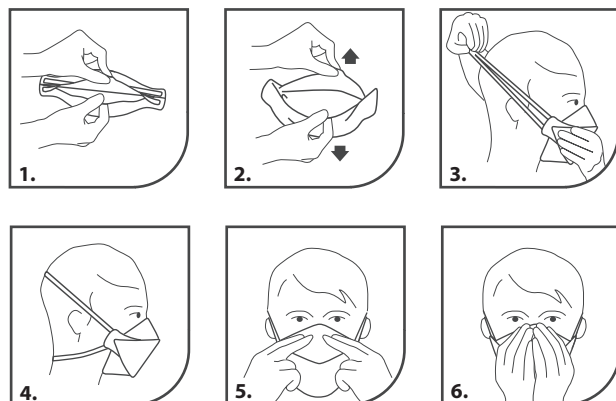

Instruction for use:

Always keep your hands clean before use.

1. With the reverse side facing up, use the loop to separate the top and bottom panels and create a shell shape. Slightly bend the nose clamp in the middle.
2. Make sure that both panels are completely unfolded.
3. Hold the half mask with one hand, open side facing the face. With the other hand grab both tapes. Hold the half mask under your chin, nose clip up, and pull the straps over your head.
4. Place the upper strap over the top of the head and the lower strap under the ears. The tapes must not be twisted. Adjust the top and bottom panels so that they fit comfortably, and the panels and chin strap must not be bent inwards.
5. Use both hands to shape the nose clip according to the bottom of the nose so that the half mask is correctly and tightly fitted. If you use only one hand to shape the clamp, the effectiveness of the mask may be reduced.
6. Before entering the workplace, the tightness test should be performed.